

CLEANASCOPE® Advantage

Transport & Short-term Storage System | **Dry & Store**





THE COMPLETE CIRCLE OF PROTECTION

As the global vanguard in infection prevention, only Cantel Medical delivers the Complete Circle of Protection, a full-value, proactive partnership dedicated to helping you remove risk, streamline operational efficiencies and optimise your success.

Clear Identification of Clean Versus Contaminated Endoscopes




Confidence is Knowing it's Clean at a Glance

The CLEANASCOPE® Advantage Transport and Short-term Storage System protects your endoscopes and other delicate equipment by minimising direct handling and reducing contact with airborne contaminants.

Its patented design creates a protective environment for each disinfected endoscope. Green (clean) and red (contaminated) colour-coded liners immediately identify clean or contaminated equipment.

Optimised Endoscope Workflow Maximises Infection Prevention

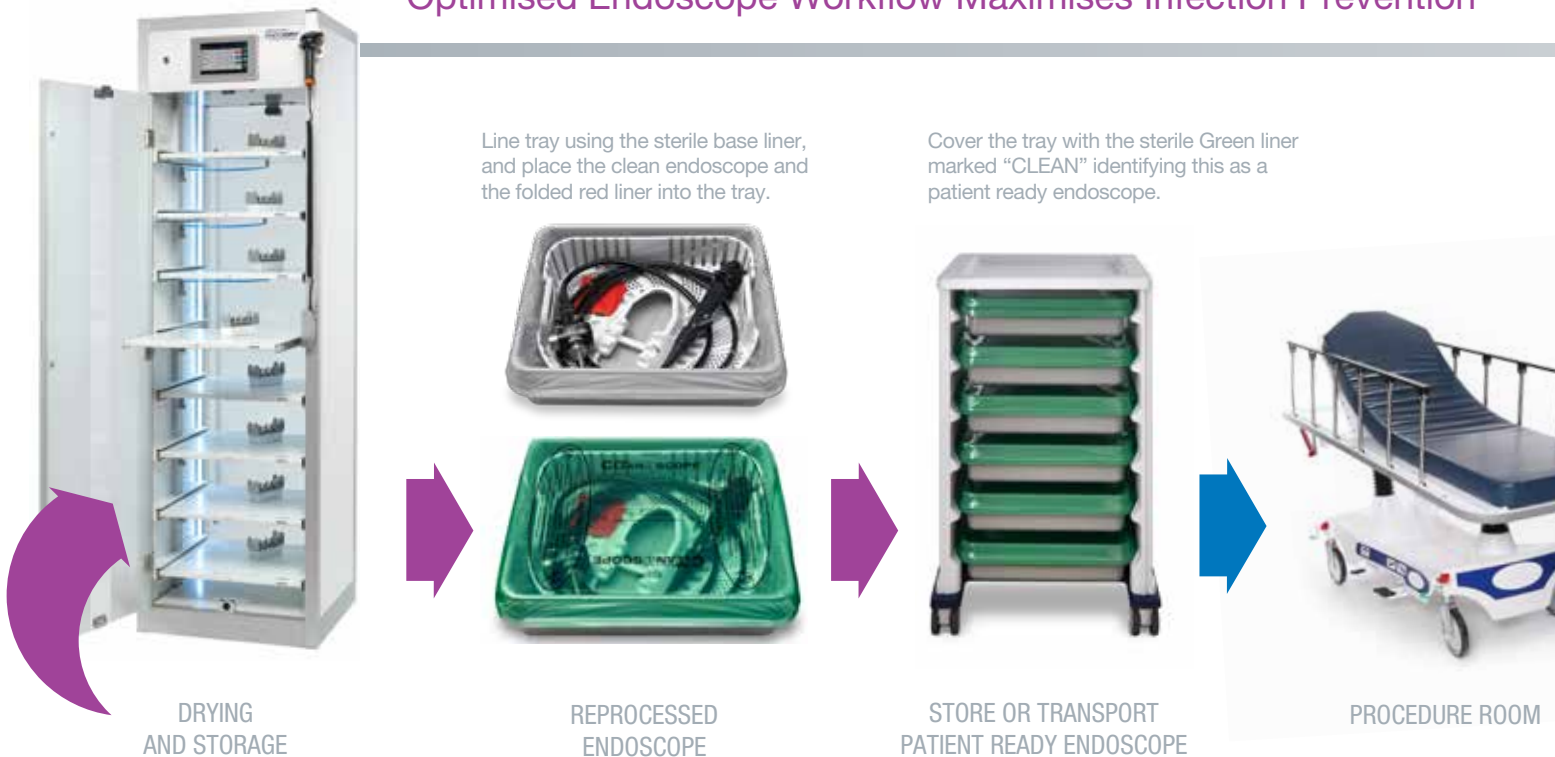



DRY & STORE

Bacteria pose significant risk to endoscopes during transport and storage. Cantel's transport, drying and storage solutions are designed to protect valuable inventory, reduce cross contamination touchpoints, eliminate moisture in the endoscope channels and control humidity. Humidity or moisture in endoscope channels is known to aid bacterial growth.

CLEANASCOPE® Advantage

Transport & Short-term Storage System | **Dry & Store**

Trays fit perfectly into the contoured cart top

Provides protected short-term storage of clean scopes and equipment

Hard lids available for added protection when transporting outside department

Available in 5, 6 and 10 tray options



Robust reusable tray easily fits large endoscopes and the ADVANTAGE PLUS® AER cassette



Three-part, single-use sterile packed liner system reduces the potential for cross-contamination

Large locking caster, wheels make transport easy and smooth

Post procedure, flush & wipe the endoscope using INTERCEPT® Detergent and place dirty endoscope inside tray. Cover tray using the Red liner for clear identification of "CONTAMINATED" endoscope.



CONTAMINATED TRANSPORT



MANUAL CLEANING



FIT SECURELY IN ADVANTAGE REPROCESSOR



CLEANASCOPE® Cart Ordering Information

Part Number	Product Description	Dimensions
103470	Five tray cart (trays included)	88.14H x 64.00W x 56.13D (cm)
103466	Six tray cart (trays included)	102.87H x 64.00W x 56.13D (cm)
103467	Ten tray cart (trays included)	161.29H x 64.00W x 56.13D (cm)



“Transport the soiled endoscope to the reprocessing area in a closed container that prevents exposure to staff, patients or the environment to potentially infectious organisms”

ASGE Quality Assurance in Endoscopy Committee, 2011 and SGNA/ESGENA Standards of Infection Control in Reprocessing of Flexible Gastrointestinal Endoscopes 2012



CLEANASCOPE® Liners & Accessories Ordering Information

Part Number	Product Description	Units Per Box
103585	Sterile Liner Set	200 Sterile liners/case
103604	Dispenser, (wall mounted)	1
103456	Tray	1
103457	Tray Lid, clear	1
CAS-1000	Advantage Cassette	1

“The sterile liner system is one of the most important reasons why many facilities are adopting the CLEANASCOPE® System – the colour coded liners take the guess-work out of endoscope reprocessing”

Libby Thomson, former Lead Nurse, Bowel Cancer Screening Centre and Nurse Endoscopist

ADVANTAGE PLUS® and INTERCEPT® are registered trademarks of Medivators Inc.
CLEANASCOPE® is a registered trademark of Cantel (UK) Limited.



Medivators BV
Sourethweg 11
6422 PC Heerlen
The Netherlands
Tel: +31.45.5.471.471
Fax: +31.45.5.429.595

Cantel France (SAS)
2 rue du Pré Fermé
31200 Toulouse (France)
Tél: +0561245275
Fax: +0561247001

Cantel Medical (Italy) S.r.l.
Via Laurentina, 169
00071 Pomezia (RM) Italia
Tel.: +39.06.9145399
Fax: +39.06.9146099

Medivators Inc.
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.763.553.3300
Fax: +1.763.553.3387